



Catheter Securement

Application of Grip-Lok WA (Wide Adhesive) for PICC Catheter Securement

Grip-Lok WA (3300M WA) is a catheter stabilization device that is strong enough to lock catheters securely in place, yet versatile enough for almost any securement situation. The adhesive on Grip-Lok WA will adhere and secure catheters made from silicone, PVC and other plastic compounds.


- Soft and flexible fabric design improves patient comfort.
- Provides superior securement for both horizontal and vertical lifting accidental line pulls.
- Simple to apply, inspect and adjust.
- Hypoallergenic, breathable and latex-free to reduce the risk of allergic reactions and skin irritation.

Prepare the skin is according to the standard hospital protocol for dressing application. Skin prep or hair removal may be required on some patients for better adhesion.

1




Slide the Grip-Lok under catheter hub and remove inner release liner to expose adhesive and press the catheter hub in place. Note: alternative technique would be to remove inner liner after securing the Grip-Lok to the skin.

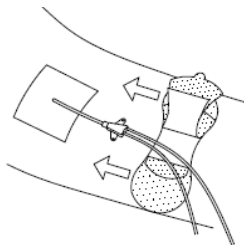

2




Pull and remove one side of the bottom release liner while holding Grip-Lok in position. Then pull and remove the other side of the bottom release liner.

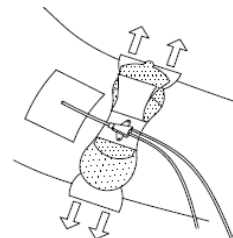

3



Secure the Velcro® loop over-strap over the catheter hub and press at the center to secure the adhesive in place.

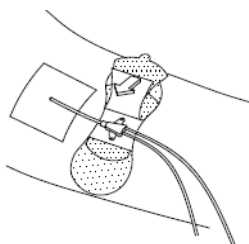

4



The Velcro® loop over-strap can be peeled back to inspect and adjust catheter hub.